



NEW!

Celebrate each season !



總代理



安德醫材有限公司

114 台北市內湖區文湖街21巷116號6F-6

TEL: 0958-777037

<https://www.adent.com.tw>


New features...



Customizable panels

4 different colors available



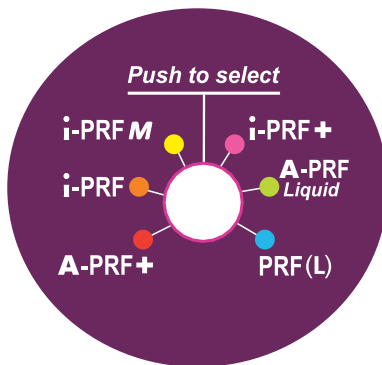



Panels are delivered
with your machine



6 PROTOCOLS



Push button
6 protocols

| | | |
|---|-----------------|--------------|
|  | 1300 rpm, 8min | ● Position 1 |
|  | 700 rpm, 3min | ● Position 2 |
|  | 700 rpm, 4min | ● Position 3 |
|  | 700 rpm, 5min | ● Position 4 |
|  | 1300 rpm, 5min | ● Position 5 |
|  | 2300 rpm, 12min | ● Position 6 |

All the values of each
this 6 protocols
are pre-set and
memorised
but not locked
so you can change*
the values if needed.

**All the protocols are
explained by
Dr. CHOUKROUN
in a video that you will find
on the USB key supplied
with your machine**

**Please refer to the User Manual
«9.5 Program modification operation»*

Additional position : There is a 7th position after PRF (L) program, it's a manual settings position. You can set here the values you want, press on the SELECT button, set the RPM values, press again on the SELECT button, set the time.

If you want to memorise the values you have setted, press the push button 7 secondes, a symbol appears next to the RPM value. The new settings are now memorized.

You can start the centrifugation by pressing START button.